



southpaw enterprises, inc.[®]

Store these instructions along with the enclosed maintenance checklist in a safe place. You may also access them on our website.

Instruction Sheet

Ladder - Home Therapy System 126020



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

- 1-Ladder
- 1-Instruction Sheet
- 1-Maintenance Checklist

MINIMUM HEIGHT REQUIREMENT:

This product is intended to hang from the Southpaw Home Therapy System's Doorway Support.

NOTE: The dowel at the bottom of the ladder is designed to be attached at only one end, so that it can be stepped on to tension and steady the ladder while being used if needed.

WORKING LOAD: 125 lbs.

MAINTENANCE CHECKLIST:

- All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.
- Check metal rings at top for excessive wear.
- Check loops at top of ropes for fraying. If fraying is found, discontinue use until repaired.
- Check metal rope clips (the metal ring under each ladder bar and at each end of ropes).
- Check wood for splinters or cracks. If splinters are found, sand smooth and refinish with lacquer. If cracks are found, discontinue use immediately.
- **Note:** If a rope clip comes loose or if a rope frays, the ladder must be returned (for repair or replacement).

STORAGE PROCEDURE:

- Store where not accessible without proper supervision.

CLEANING INSTRUCTIONS:

- Wipe with a damp cloth.
- Do not immerse in water
- Dry before using

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



**Southpaw
Enterprises Inc.**[®]

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047

Toll Free - (800) 228-1698

www.southpawenterprises.com